# TEST-8

**INDICE****TABLE OF CONTENTS**

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#### 1.1. Notice

##### **Please Note.**

The Italian version of this manual is the master from which translations derive.  
In case of any discrepancy, the binding version is the Italian text.

**You are advised to carefully read these instructions for installation, use and maintenance of the equipment before operating it, in order to avoid misuse or damage of any kind.**

#### 1.2. Meaning of the Pictograms and Terms Used in This Manual

##### **Warning!**

This box contains safety warnings.

##### **Warning hot surface!**

Hot surface, do not touch.

##### **Warning electricity!**

This box contains electric safety warnings.

##### **Please note**

This box contains operational information

##### **Warning biohazard!**

This box contains biological safety warnings

#### 1.3. Warnings

These instructions for use are drafted for the TEST-8.

The device is designed for the incubation of biological indicators (*Bacillus Stearothermophilus*) used for the verification of autoclaves or ethylene oxide sterilizers.

This device must be used by trained healthcare personnel in the clinic or the dental office.

The manufacturer of the device is:

**Silfradent S.r.l.**

Via G. Di Vittorio N°35/37

47018 S. Sofia (FC), ITALY

Tel. +39 0543 970684

Fax +39 0543 970770

WEB: [www.silfradent.com](http://www.silfradent.com) E-mail: [info@silfradent.com](mailto:info@silfradent.com)

**Silfradent S.r.l. congratulates and thanks you for buying this equipment.**

It is important that you make sure that the package contains the guarantee certificate and that this is filled in and stamped by the retailer.

After removed the device from package, place it on a plane and check for no damages due to transport, otherwise immediately advise the retailer.

Silfradent S.r.l. has drafted these instruction for use with the aim to help the user and facilitate the start up and operation of the equipment.

A copy of this manual is provided together each device.

Compliance with the instructions contained in this technical document is necessary for correct operation.

The use of the equipment for uses different from those indicated in these instructions and in the brochures is absolutely forbidden.

Non-compliance with the instructions herein contained results in the immediate termination of the guarantee.

**Please note!**

For any request please always indicate:

the date of purchase, model and serial number of the equipment that are on the data label located on the bottom side of the device.

The removal of the data label will result in the expiration of the guarantee.

## 1.4. Safety Warnings

Before use, always check the operation and condition of the device and the power cord.

If the equipment or the power cord is damaged, do not use it and contact the technical assistance service.

Before leaving the equipment unattended, turn off the main switch and, if necessary, disconnect the power supply cable from the mains socket.

**Warning electricity!**

To avoid the risk of electric shock this device must only be connected to a socket equipped with a protective earth connection.

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Always remove the power cord before carrying out any maintenance or cleaning operations.

Never pull out or handle the power cord with wet hands

Never use solvents or flammable materials near this or other electrical appliances.

Always check the power cord before use.

During cleaning or disinfection, make sure that the liquid does not enter the inside of the device.

Do not immerse the APAG or pour liquids over it, it is made up of electrical components. If the device gets wet, dry it and store it in a dry place for at least one day before using it.

**Please note!**

To isolate the device from mains, remove the power cord from the outlet or from the connect-or in the back of the equipment.

**Warning hot surface!**

Never touch the surface of the aluminum heater during operation.

**Warning!**

No modification of this equipment is allowed.

## 1.5. Characteristics of the TEST-8

Nominal voltage (Vac)	230 Vac +/-10%
Nominal frequency (Hz)	50/60
Rated power (W)	50
External fuses	
Glass type Ø 5 x 20mm	1x T 0,4 A L 250 V
Insulation class	1
User interface	Un tasto tre LED
Internal fuses	
(replaceable only by service personnel)	1 x T 0,1 A L 250 V
Dimensions of the device L x W x H (mm)	180 x 175 x 80
Weight (kg)	1
Environmental operating conditions	Indoor use, Altitude up to 2000 m a.s.l., Humidity up to 80% Ambient temperature between 2 and 40 ° C

## 1.6. Waste disposal

The device must not be considered as solid urban waste, it is electrical and electronic equipment (WEEE).

It can be delivered to centers for the selective collection of electrical and electronic equipment or delivered to the distributor when purchasing an equivalent product.

For more information, consult the nearest Silfradent S.r.l. office.

### Warning biohazard!

Disposal of biological indicators vials must be done in accordance with their manufacturer's instructions.

## 1.7. Guarantee

Silfradent S.r.l. guarantees to the purchaser that it will repair or - to its sole discretion - replace any part which is defective with regard to construction or materials, in normal conditions of use, for the period of one year from the date shown on the data label on the bottom of the equipment.

Silfradent S.r.l. is not liable for any damage depending on:

- External causes (poor quality of liquids or incorrect installation)
- Use of incorrect information
- Use of the equipment not in compliance with the regulations in force.
- Unsuitable repair.

The changes made by unauthorized third parties void any equipment guarantee.

Repair and maintenance of the device are the responsibility of the Silfradent's technicians or the authorized personnel.

The guarantee is provided ex-factory Silfradent S.r.l.

The guarantee does not cover defects or damage arising from:

- Falls, damage due to excessive heat, spills or penetration of liquids, natural disasters.
- Use not in compliance with the instructions of this manual.
- Incorrect or unsuitable maintenance by the user.
- Unauthorized opening of the external casing.
- Unauthorized tampering or changes.
- Operation not in compliance with the environment specifications indicated for the product.
- Use of non original accessories.
- Cleaning with unsuitable products.

### Nota bene.

Only original spare parts must be used to repair the TEST-8

The replacement of components must be done with parts supplied or recommended by Silfradent S.r.l.

## 1.8. Transport and storage

In the case of return shipment, wrap the product correctly by using the original package, if possible.

It is advisable to insure the package.

The customer has full responsibility for damages arising from unsuitable packaging.

Transport and storage of the equipment **MUST BE CARRIED OUT** into original packaging at a temperature range of -40 to +70 °C ", with relative humidity 10 to 90% and pressure ranging between 500 and 1060 HPA.

## 2. Description

### 2.1. General description

The TEST-8 biological incubator is a device designed for the incubation of biological indicators that must be used periodically to check the efficiency of ethylene oxide sterilizers and autoclaves.

It is designed to work at temperatures of 37° C or 57° C, has eight holes for inserting the test vials, inside an aluminum heating unit.

It has to work on a horizontal plane.

### 2.2. Equipment of the TEST-8

- 1 **TEST-8**
- 1 **Power cord**
- 1 **Certificate of guarantee**

- 1 **Declaration of conformity**
- 1 **Instruction for use**

## 2.3. Data label

 <b>Silfradent srl</b> via G. di Vittorio 35/37 S.SOFIA - FORLÌ - ITALY	Address of the manufacturer	230 Vac    50/60 Hz    50VA Fuse T 400 mA    250V Rated voltage and nominal power, and type of fuses
 YYYY-MM	Month of production	
<b>REF TEST-8</b>	Device code	 Do not dispose of as solid urban waste, use collection centers.
<b>SN XXXX-YYY- ZZZ</b>	Serial number	
<b>Made in Italy</b>	Country of manufacture	
<div> <div> <b>REF TEST-8</b>  <b>SN XXXX-YYY- ZZZ</b>            230 Vac    50/60 Hz    50VA            Fuse T 400 mA    250V            YYYY-MM   <b>Silfradent srl</b>            via G. di Vittorio 35/37            S.SOFIA - FORLÌ - ITALY            Made in Italy         </div> <div>        </div> </div> <p><b>Data label TEST-8</b></p>		 The equipment meets the EU compliance characteristics.
		 Refer to the manual and safety information
		 Read the instructions for use
		 Hot surface (this symbol is in the front part of the device)

## 3. Functioning

### 3.1. Description of the front panel


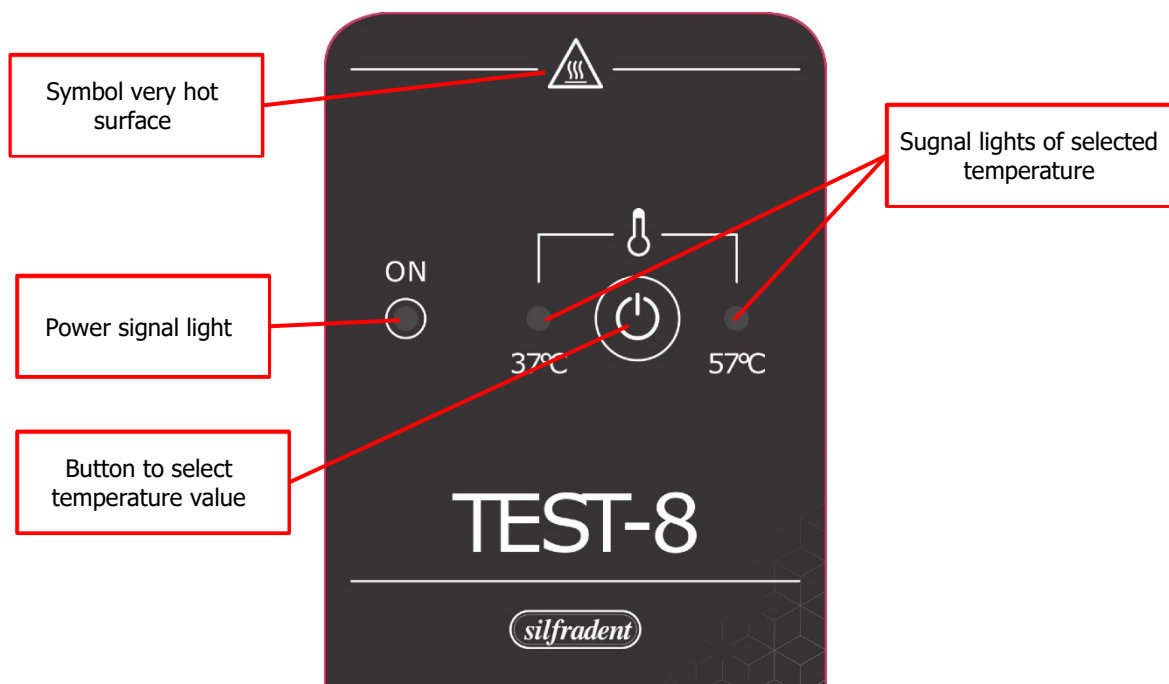
Fig. 2 front panel

## 3.2. Instruction for use

When turn on the device the LED **ON** lit

To start operation see below

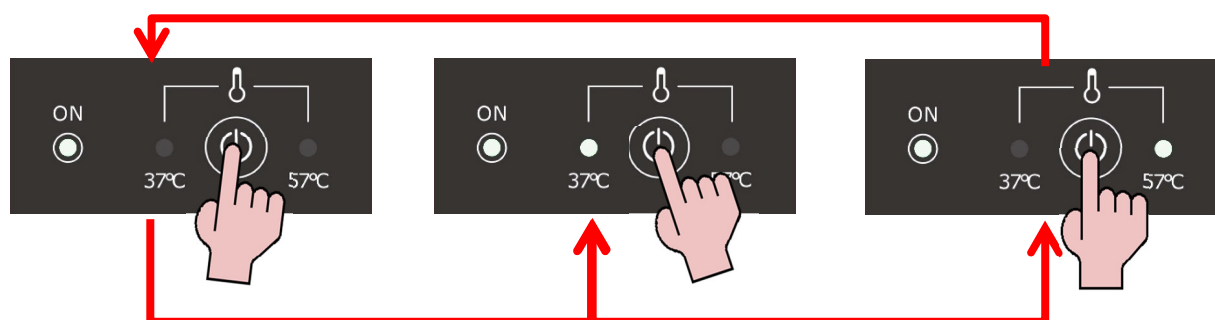Press the button once to select the temperature 37°C, the heating starts and the 37°C LED starts blinking.

When the temperature is reached it stops flashing and stays on.

Press the button the second time, the temperature 57°C is selected, the heating starts and the LED 57°C starts blinking.

When the temperature is reached it stops flashing and stays on.

Press the button the third time, the heating unit turns off. Only the power light remains lit (ON)

- Insert the biological indicator vials into the sterilizer together with the sterilization load
- At the end of the sterilization cycle, take the biological indicator from the sterilizer and let it cool for a few minutes (typically 8/10).
- Activate the vials, compressing them until the internal ampoule breaks, so that the disk containing the spores remains immersed in the culture broth.
- Do the same with an unsterilized vial to use as a control vial
- Connect the power cable of the TEST-8 to the power socket, the power indicator lights up.
- Insert the vials of the biological indicator taken from the sterilizer and the control one into the seats of the TEST-8 heating unit (see figure above).
- Press the temperature selection button to choose the required incubation temperature.
- The selection of the temperature activates the heating cycle and the light of the selected temperature starts to blink; the temperature of the heating group rises.
- Wait for the time necessary for the spores to incubate; it is indicated by the manufacturer of the biological indicators.
- At the end of the time required to switch off the heating, press the selection button twice if 37 ° C is selected or once if 57 ° C is selected
- Compare the colors of the test vials with the color of the control one
- If during the necessary incubation time the incubator is found to be stopped (temperature LEDs off) the test is not considered valid and must be repeated.

During the test, carry out periodic checks to check if there are any variations in the color of the samples.

## 3.3. Maintenance

### Please Note!

Do not use abrasive cleaners..

### Warning electricity!

During cleaning or disinfection, pay attention that no liquid enters the device.

Never use flammable products to clean the device or its accessories.

Never carry out any maintenance or cleaning without removing the power cord.

The external surface can be cleaned with a soft cloth moistened with water and neutral detergent, dry with a soft dry cloth.

The enclosure is not waterproof, liquid can penetrate during cleaning.

Do not use abrasive products on the external surface of the machine, it could get scratched.

Wait for the device to cool down before cleaning it.

## 4. Troubleshooting

### Warning!

Only Silfradent S.r.l. authorised technicians have access to the internal parts of the equipment.

### Warning electricity!

Do not try to open **the TEST-8**, dangerous high voltage is present inside the equipment.  
Before carrying out any maintenance or repair, always remove the power cord.

### Please Note.

The wiring diagrams and technical support information are available to the personnel responsible.  
For repairs and additional information is required to contact Silfradent S.r.l.

PROBLEM	POSSIBLE CAUSE	SOLUTION
The ON LED does not light.	The power cord is not plugged in properly.	Check that both the socket towards the appliance and the mains plug are inserted correctly.
	The fuse in the power connector on the back of the device has blown	Replace the fuse with an identical one, the value is shown in par. 1.5
	The internal fuse has blown	Contact the technical assistance of Silfradent S.r.l.
The cycle was interrupted during operation	There was an interruption in the electricity supply	The result of the test in progress is to be considered invalid.

Via G. Di Vittorio N°35/37 47018 S. Sofia (FC), ITALY  
TEL +39 0543 970684 FAX +39 0543 970770  
WEB: [www.silfradent.com](http://www.silfradent.com)  
E-mail: [info@silfradent.com](mailto:info@silfradent.com)